GENEMED

GM501 Flush

Material included in the package:

Product code: 4 GM 501F-20
1 x 20 ml GM501 Flush
Product code: 4 GM 501F-50
1 x 50 ml GM501 Flush
Product code: 4 GM 501F-100
1 x 100 ml GM501 Flush
Product code: 4 GM 501F-250
1 x 250 ml GM501 Flush
Product code: 4 GM 501F-500
1 x 500 ml GM501 Flush

Material not included in the package:

Incubator (no CO₂)
Petri-dishes
LAF bench (ISO 5 environment)
Microscope
Test tubes
Rubber free syringe

Product specifications and quality control:

All raw materials are of highest available purity (European Pharmacopoeia and/or USP standard) if applicable.
A certificate of analysis is available for each batch upon request from our website with respective lot number.
The MSDS for GM501 Flush is available upon request and can also be downloaded from our website.
GM501 Flush is manufactured and tested according to the following specifications:

pH (at 37°C)	7.20-7.60 (Release criteria: 7.20-7.40)
Osmolality (mOsm/kg)	270-290
Sterility	sterile - SAL10 ^{−6} (Sterility Assurance Level)
Endotoxins (EU/ml)	≤ 0.25
Mouse Embryo Assay (MEA)	≥80
(1-cell assay, blastocysts after 96 h in %, 60 min exposure at zygote stage)	

EN: Intended use/intended users:

Cell culture medium for ovarian oocyte pick-up.
GM501 Flush is a ready-to-use medium for flushing the ovarian follicles during aspiration and/or oocyte pick-up intended for extra corporeal fertilization procedures.

The intended users are IVF professionals (lab technicians, embryologists or medical doctors).

Composition:	NaCl, KCl, KH ₂ PO ₄ , MgSO ₄ ·7H ₂ O, NaHCO ₃ , CaCl ₂ ·2H ₂ O, D(+)-Glucose anhydrous, Na-Lactate (50 % solution), Na-Pyruvate, EDTA, Alanyl-Glutamine, water, non-essential and essential, Amino Acids, HEPES, 2.5 IU/ml Heparin.
---------------------	--

Gentamicin sulfate can be added by the user (10 mg/liter).

GM501 Flush needs to be warmed at 37°C over night before use (no CO₂ with closed lid). GM501 Flush is HEPEs-buffered. Incubation in a CO₂-incubator will lower the pH.

Oocyte collection:

Keep GM501 Flush medium at 37°C or as close to this as possible.
Take about 20-50 ml medium at a time, using a rubber free syringe.
Assemble medium and tubing set according to the manufacturers instructions.
Attach syringe containing GM501 Flush medium to the appropriate Luer, allowing connection to the lumen of the needle.
The person holding the syringe should try to apply warmth to it by holding it in the palm of the hand covering as much of the syringe surface as possible.
During aspiration of follicles, if an ovum is not extracted after a follicle has been emptied, flushing is indicated. The collapsed follicle must be refilled with GM501 Flush, after which it may be aspirated once again. This may be done repeatedly, after refilling the syringe with fresh aspiration medium each time, until an ovum is recovered or the surgeon decides to abandon the follicle (after three attempts at flushing it may be sensible to move on, but the ultimate decision rests with the surgeon).

Precautions and warnings:

All human organic material should be considered potentially infectious. Therefore, handle all specimens as if capable of transmitting HIV or hepatitis.
Always wear protective clothing when handling specimens.
Always work under strict hygienic conditions (ISO 5 environme, e.g. LAF-bench) to avoid possible contamination.
GM501 Flush contains heparin which is derived from porcine intestinal mucosa.
Only for the intended use.
This facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Pre-use checks:

Do not use the product if it becomes discoloured, cloudy, or shows any evidence of microbial contamination.
Do not use the product if the seal of the container is opened or defective when the product is delivered.

Storage instructions and stability:

The shelf life is 6 month from time of manufacture.
Store between 2-8°C.
Do not freeze before use.
Keep away from sunlight.
The product can be used safely up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8°C.
Stable after transport (max. 5 days) at elevated temperature (≤ 37°C).
Content cannot be re-sterilized after opening.
Do not use after expiry date.

 	
----------------	----------------

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