GENEMED

GM501 Mineral Oil

Material included in the package:

- Product code: 4 MO 0100
- 1 x 100 ml GM501 Mineral Oil
- Product code: 4 MO 0500
- 1 x 500 ml GM501 Mineral Oil

Material not included in the package:

- Incubator (at 37°C)
- Plain dishes
- LAF bench (ISO 5 environment)
- Microscope
- Test tubes
- Cell culture media

Product specifications and quality control:

- Pharmaceutical grade mineral (paraffin oil).
- A certificate of analysis is available for each batch upon request from our website with respective lot number.
- The MSDS for GM501 Mineral Oil is available upon request and can also be downloaded from our website.
- GM501 Mineral Oil is manufactured and tested according to the following specifications:

Density (g/ml)	0.83-0.86
CAS code	< 30
Viscosity (at 30°C/cP)	< 30
Sterility	sterile - SAL10% (Sterility Assurance Level)
Endotoxins (EU/ml)	< 10.0
Mouse Embryo Assay (MEA)	≥80
(1-cell assay, bioactivity after 120 h in % , 120 h exposure at zygote stage)	
Peroxide Value (POV) (mEq/kg)	< 0.1

EN: Intended use/intended users:

- GM501 Mineral Oil is a ready-to-use oil for covering the medium during IVF/ICSI treatment. GM501 Mineral Oil protects the medium from evaporation and thereby stabilizes pH and temperature.
- The intended users are IVF professionals (lab technicians, embryologists or medical doctors).

Compositon:
Mineral Oil
CAS code: 8042-47-5
EP Monografija: 0240

General Information:
GM501 Mineral Oil (density 0.83 - 0.86 g/ml) is pre-washed twice with ultra-pure water. GM501 Mineral Oil should be pre-incubated for 5 hours with the lid closed at 37°C

Instructions for use:
GM501 Mineral Oil is a pre-washed ready-to-use oil, so no further preparations are necessary.
We recommend preparing the dishes with oil overlay the day before use. If using bicarbonate buffered media incubate the dishes in a CO₂ incubator at 37°C to ensure warming and saturation with gas.

Any procedures described herein are recommendations only. Each laboratory needs to establish and validate its own procedures for medium preparation and use.

Precautions and warnings:

- Handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.
- Always work under strict hygienic conditions (e.g. LAF-bench ISO Class 5) to avoid possible contamination.
- For the intended use.

The oil may appear cloudy if removed from the incubator and allowed to stand at room temperature for extended periods of time. Such cloudiness usually disappears if the oil is re-equilibrated by replacing it in the incubator for several hours, if it does not disappear discard the product.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Pre-use checks:

- Do not use the product if it becomes discoloured, cloudy, or shows any evidence of microbial contamination.
- Do not use the product if seal of the container is opened or defective when the product is delivered.

Storage instructions and stability:

- The shelf life is 18 months from time of manufacture.
- Store between 15-25°C. Storage temperature between 2-15°C is also possible. This might cause some turbidity which disappears when the oil is warmed.
- Keep the product in the original packaging or shield from light white not in use.
- Do not freeze before use.
- Do not expose to direct sunlight.
- Avoid excessive sources of irradiation.
- The product can be used safely for up to 28 days after opening the container when sterile conditions are maintained and the products are stored at room temperature.
- Stable after transport (max. 5 days) at elevated temperatures (≤ 37°C).
- Do not use after expiry date.
- Cannot be reutilized after opening.

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