Material included in the package:

- GM501 SpermActive - 4 GM 501 SA-020
- 1 x 20 ml GM501 SpermActive
- Product code: 4 GM 501 SA-050
- 1 x 50 ml GM501 SpermActive

Material not included in the package:

- CO₂-incubator at 37°C
- Petri-dishes
- LAF bench (ISO 5 environment)
- Microscope
- Test tubes

Product specifications and quality control:

- All raw materials are of highest available purity (European Pharmacopoeia and/or USP standard) if applicable.
- A certificate of analysis is available for each batch upon request from our website with respective lot number.
- The MSDS for GM501 SpermActive is available upon request and can also be downloaded from our website.
- GM501 SpermActive is manufactured and tested according to the following specifications:

pH (at 37°C, 6% CO ₂)	7.20-7.50
Osmolality (mOsm/kg)	270-290
Sterility	sterile - SAL10 ³ (Sterility Assurance Level)
Endotoxins (EU/ml)	< 0.25
Mouse Embryo Assay (MEA) (1-cell assay, blastocysts after 96 h in %, 60 min exposure at zygote stage)	≥ 80

EN: Intended use/Intended users:

- GM501 SpermActive is a ready-to-use medium designed for all human sperm preparation, sperm washing and swim up techniques like density-gradient centrifugation as well as for testicular tissue.
- The intended users are IVF professionals (lab technicians, embryologists, or medical doctors).

Composition:
NaCl, KCl, KH₂PO₄, MgSO₄, 7H₂O, NaHCO₃, CaCl₂·2H₂O, D(+)-Glucose anhydrous, Na-Lactate, Na-Pyruvate, EDTA, Alanyl-Glutamin, water, non-essential amino acids, 15 mM HEPES, human serum albumin (5.0 g/liter), Gentamicin (10.00 mg/liter), Phenolred

Instructions for use:
• GM501 SpermActive must be equilibrated overnight in a humidified CO₂-incubator (at 5-7 % CO₂, 37°C).

Precautions and warnings:
GM501 SpermActive contains the antibiotic Gentamicin Sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitive to this antibiotic.

• Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Since this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infection agents cannot totally be excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin, manufactured to European Pharmacopoeia specifications by established processes.

• Therefore, handle all specimens as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling specimens.

• Always work under strict hygiene conditions (ISO 5 environment, e.g. LAF-bench) to avoid possible contamination, even when GM501 SpermActive Medium contains Gentamicin.

• Only for the intended use.

• The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Pre-use checks:
• Do not use the product if it becomes discoloured, cloudy, or shows any evidence of microbial contamination.

• Do not use the product if seal of the container is opened or defective when the product is delivered.

Storage instructions and stability:
• The shelf life is 6 month from time of manufacture.

• Store between 2-8°C.

• Do not freeze before use.

• Keep away from (sun) light.

• The product can be used safely up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8°C.

• Stable after transport (max. 5 days) at elevated temperature (≤ 37°C).

• Content cannot be re-sterilized after opening.

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GYNEMED GmbH & Co. KG